Quick Start Guide **EN**

Schnelleinstieg **DE**

Guida rapida **IT**

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Quick Start Guide

UNEEDG™ EpiSight Analyzer

Version number 2.1

Manufacturer:
AIT - Austrian Institute of Technology GmbH;
Giefinggasse 4; 1210 Vienna; Austria
Date of manufacture: 2023

Use-by date:
Windows 10: 10/2025
Windows Server 2019: 01/2024

Notified Body:
mdc medical device certification GmbH;
Kriegerstrasse 6; 70191 Stuttgart; Germany

Medical device

Caution: Please read paragraphs marked with this symbol carefully as they contain important safety information.

(01)09120109830177

Issuing date of this document:
2023-07

Table of Content

Table of Content.....	3
Warnings and precautions.....	4
Contraindications.....	5
Possible adverse effects.....	5
Quick Start Guide	6
Installation and license activation	7
Patient Database	9
Trending	10
Spectrogram.....	11
Report	12
Keyboard shortcuts.....	13

Warnings and precautions

UNEEDG™ EpiSight Analyzer is CE-certified as a medical device. If your country requires any other approval for medical devices you must not use UNEEDG™ EpiSight Analyzer.

The modules from UNEEDG™ EpiSight Analyzer for the automatic analysis of EEG cannot replace the examination by the physician. As for any other automatic procedure, there can be inaccuracies during the analysis with UNEEDG™ EpiSight Analyzer. The original EEG still needs to be used for the evaluation and the results of UNEEDG™ EpiSight Analyzer need to be confirmed based on the unaltered raw EEG trace.

UNEEDG™ EpiSight Analyzer does not provide any diagnosis or diagnostic recommendation, conclusion or prediction of the patient's state. It remains the responsibility of the physician to decide on the diagnosis or to induce treatment.

The use of the software cannot replace the real-time surveillance by medical staff particularly of the vital functions. The patient's safety remains the responsibility of the medical professional.

The maximum length for continuous EEG recording that can be reviewed and analyzed with the EEG viewer is 5 years. If a recording is longer than 5 years UNEEDG™ EpiSight Analyzer will stop processing.

For UNEEDG™ EpiSight Analyzer seizure detection, detection of rhythmic pattern, aEEG and background frequency the EEG of at least three subcutaneous electrodes must be available.

UNEEDG™ EpiSight Analyzer seizure detection cannot be used as an alarm system, since there may be a delay of several minutes between the clinical onset of a seizure and the display of the detected pattern. Some EEG-events with epileptic seizures that have an electrographic correlate might be missed. For this reason UNEEDG™ EpiSight Analyzer can only support but can not replace the qualified expert in the real-time monitoring.

UNEEDG™ EpiSight Analyzer detection of rhythmic pattern, aEEG or background frequency included in UNEEDG™ EpiSight Analyzer should not be used as an alarm system, since the visualization of patterns could be delayed by up to several minutes.

Some events, where patterns defined in the Standardized Critical Care EEG Terminology of the American Clinical Neurophysiology Society are visible in the EEG, might be missed. All measures of UNEEDG™ EpiSight Analyzer must be evaluated in combination with the original raw EEG and must not be the only basis for a diagnosis, treatment or change of treatment.

Contraindications

There are no known contraindications for the use of UNEED™ EpiSight Analyzer.


Possible adverse effects

There are no known possible adverse effects when using UNEED™ EpiSight Analyzer.

Quick Start Guide


Installation and license activation

1. Download the installer for UNEEG™ EpiSight Analyzer.
2. Start the installer by double-clicking the downloaded executable.
3. The installer will guide you through the installation process.
4. After the installation the following window will pop up: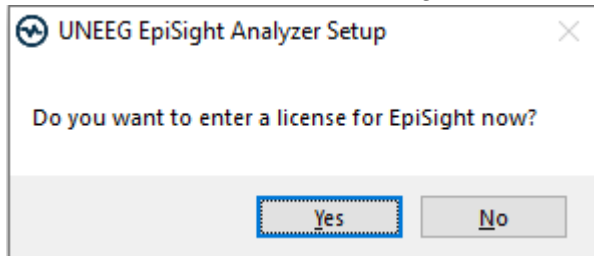


If you click “Yes” you will proceed at step 9.

5. Once the installation has finished, start the EpiSight Analyzer by clicking on the icon on your desktop.
6. You can add new patients to the database and import data.
7. The following steps are just necessary if you have no valid License activated so far. In order to open the patient’s EEG, you will need an active license.
8. Click on “Open patient”, the following window will pop up: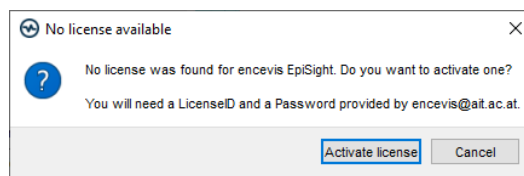

9. Press “Activate license”, the License Tool will pop up.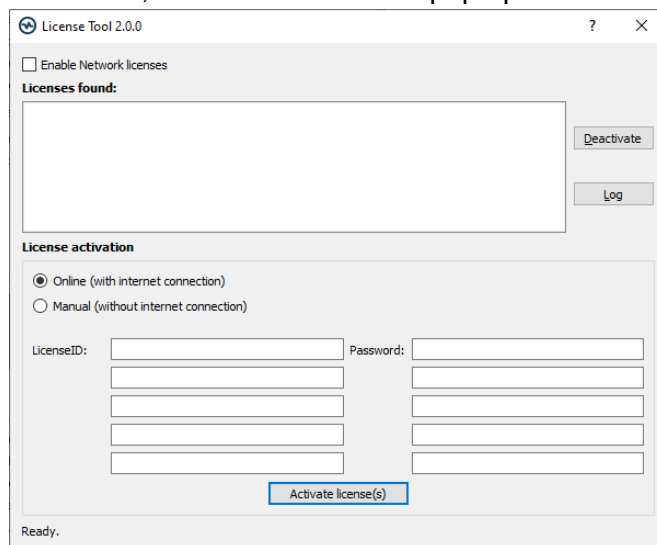

10. If you are connected to the internet, select „Online“ as activation mode. Otherwise, select “Manual” and contact your distributor in order to get the password for the manual activation.
11. Enter your LicenseID and Password.
12. Press “Activate license(s)”.

Patient Database

Starting UNEEG™ EpiSight Analyzer will open the patient database window.


The screenshot shows the UNEEG EpiSight Analyzer interface. The top menu bar includes: Search Patient, Clear search, Select columns, Add new patient, Check connection, Change settings, Open Help, Open data, Generate report, and Import data. The main window is divided into two panes. The left pane, titled 'Patients', contains a search bar and a table with columns: Patient ID, Last name, First name, Birth date, Connection, and Open. The right pane, titled 'Details for Doe John - Unconnected', displays patient information (Patient ID: JM19700101, Last name: John, First name: Doe, Birth date: 1970-01-01, Sex: unselected, Seizure type(s): unselected, Treating physician: unselected, Implant serial no.: E 02, Location: Unknown), a list of UNEEG devices, a comment field, and data information (Start of recording: 2018-02-11 08:54:23, Current end of recording: 2018-05-16 15:54:49, Available amount of data: 66 days 13 hours, New data since last review: No last review available, Reviewed until: No last review available). At the bottom, there is a data overview timeline showing data available (grey), no data available (white), and data analyzed (blue). Annotations with arrows point to various features: 'Search Patient' points to the search bar; 'Clear search' points to the 'Clear search' button; 'Select columns' points to the 'Select...' dropdown; 'Add new patient' points to the 'Create new patient' button; 'Check connection' points to the 'Check connection' button; 'Change settings' points to the 'Settings' button; 'Open Help' points to the 'Help' button; 'Open data' points to the 'Open patient' button; 'Generate report' points to the 'Report' button; 'Import data' points to the 'Import manually' button; 'Edit details' points to the 'Edit details' button; 'Add comment' points to the comment field; and 'Read data information' points to the data information section.

The patient database offers the following functionalities:

- Search for a patient by ID, name or date of birth
- Select the columns shown in the patient list
- Add a new patient
- Edit the patient's information
- Get the information about the EEG data available for each patient
- Open the patient's EEG
- Generate a report
- Connect to UNEEG™ ProConnect
- Import data by connecting to UNEEG™ ProConnect
- Import data manually by selecting a folder
- Change settings
- Open help

Trending

Clicking on “Open patient” will open the trending window.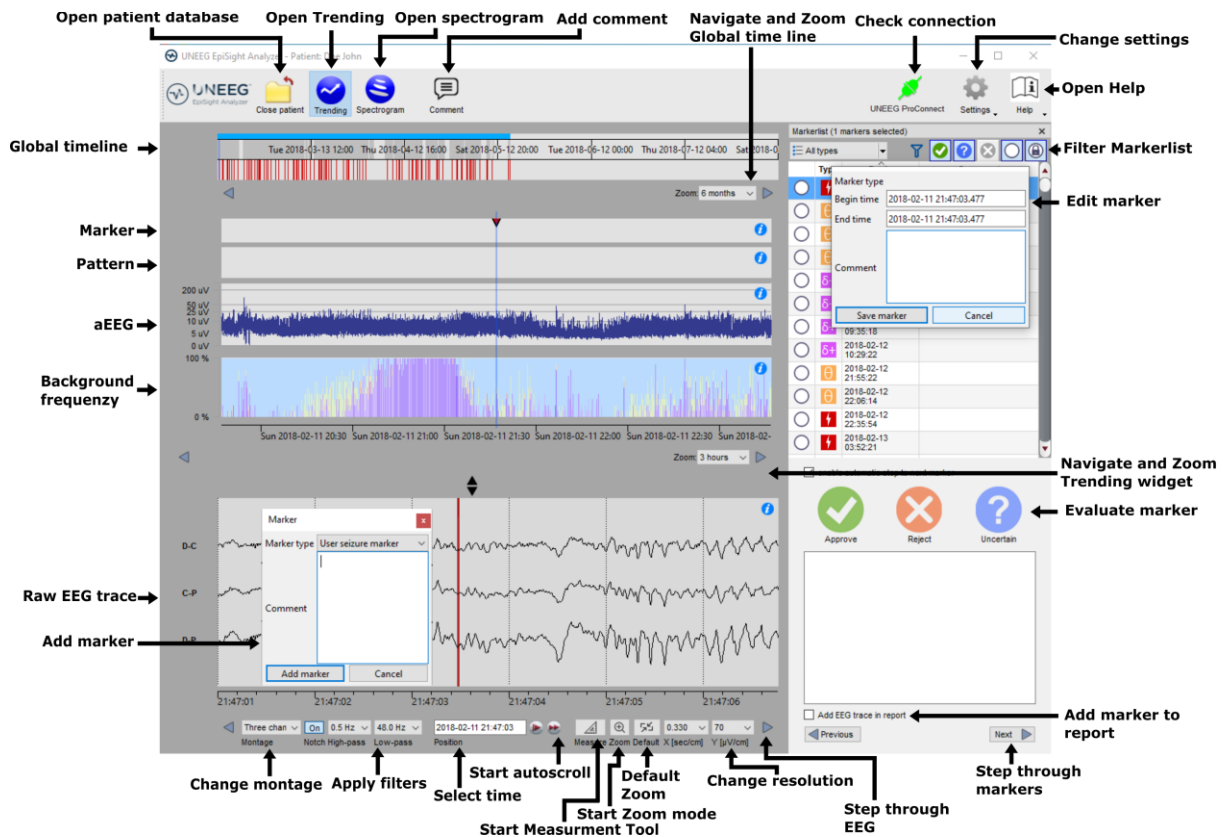


The trending includes the following functions and traces:

- Global timeline with an overview over all available data
- Zoom in the global timeline and in the analysis windows over a drop-down menu or the mouse wheel
- Seizure and other markers in a trace and in a list on the right side of the window
- Rhythmic pattern markers
- Amplitude-integrated EEG (aEEG)
- Background frequency
- EEG plot synchronized to the traces above
- Change montage
- Apply notch, high-pass and low-pass filters
- Change resolution in time and voltage
- Apply standard zoom
- Start zoom mode
- Start measurement Tool
- Start autoscroll
- Add markers
- Review and edit markers
- Filter marker list by marker type and by status

Spectrogram

Clicking on the icon “*Spectrogram*” will show the spectrogram traces.

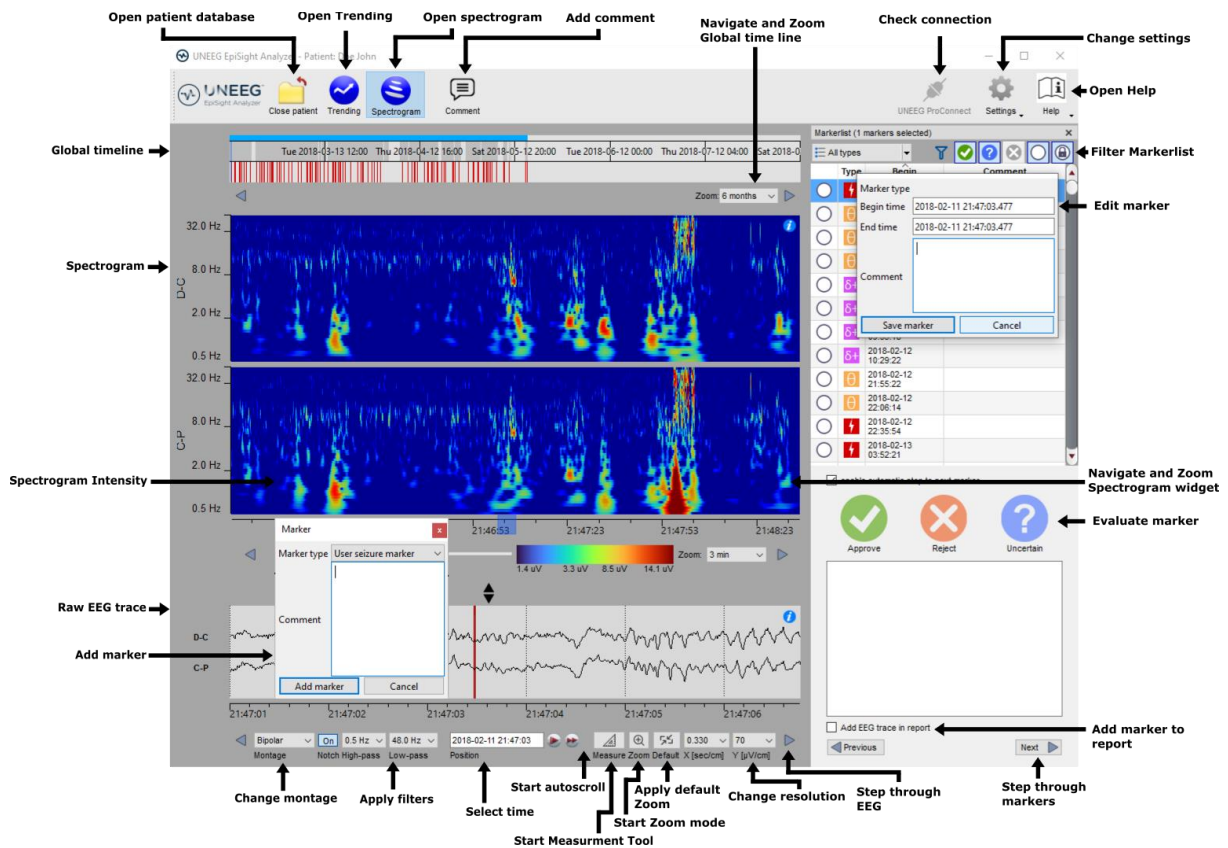The spectrogram adds the following functionalities:

- Display of the spectrogram of the channels D-C and C-P.
- Get the estimated frequency in the spectrogram trace
- Adjust the color settings by changing the maximum intensity shown.

Report

Clicking on “Report” will open the report window.

The screenshot shows the 'Report for patient ID e02' window. Annotations include:

- Automatically generated content:** Points to the Patient information, Statistical information, Report Details, and Seizure periods sections.
- Clear:** Points to the 'Clear All' button in the top right.
- Export to PDF:** Points to the 'Create PDF' button in the top right.
- Fill out:** Points to the 'Indication for testing', 'Summary of the findings', 'Diagnostic significance', and 'Clinical comments' text areas.
- Paste automatically comment:** Points to the 'Paste from EpiSight comment' button.

The report content includes:

- Report for patient ID e02:** Fields for 'Reported by', 'Reference period start/end', and 'Current period start/end'.
- Patient information:** Date of birth (1970-01-01), Age at study time (49), Sex (unselected), Treating physician (unselected), Seizure types (Implant ID: E 02, Location: Unknown).
- Statistical information:** Hours recorded (967 630), Usage total (84% 56%), Usage day (7am - 10pm: 84% 56%), Usage night (10pm - 7am: 83% 55%), Electrographic seizures (0 0), Electrographic seizure rate per 24 hours (0.00 0.00), Electrographic seizure rate per 24 hours of recording (0.00 0.00), Patient-reported events (0 0).
- Report Details:** Section for 'Seizure periods' with a heatmap showing seizure activity over time (2018-02-11 to 2018-05-02).

The report offers the following functionalities:

- Enter additional clinical information including:
 - Indication for testing
 - Summary of the findings
 - Diagnostic significance
 - Clinical comments
- Generate content automatically including:
 - Patient information
 - Statistical information
 - Overview of seizure periods over time
 - Seizure periodicity plots
 - Seizure rate over time
 - Examples of seizures
- Paste the EpiSight comment
- Export to a pdf file

Keyboard shortcuts

The following table shows a summary of keyboard shortcuts available

Modifier	Key	Function
		EEG Plot area
	Arrow left	Scroll EEG-Plot one page to the left
	Arrow right	Scroll EEG-Plot one page to the right
Shift	Arrow left	Scroll EEG-Plot one second to the left
Shift	Arrow right	Scroll EEG-Plot one second to the right
	Home	Scroll EEG-Plot to the beginning of the recording
	End	Scroll EEG-Plot to the end of the recording
Ctrl	+	Zoom one step into the EEG Area
Ctrl	-	Zoom one step out of the EEG Area
Ctrl	0	Zoom EEG Area to default value
Shift	MouseWheel	Zoom at the cursor position in EEG Area
		Marker List
	Arrow up	Scroll one row up in the marker list
	Arrow down	Scroll one row down in the marker list
Shift	Arrow up	Expand selection with next marker up
Shift	Arrow down	Expand selection with next marker down
Ctrl	A	Select all marker currently visible in the marker list
Ctrl	C	Copy selected marker to the clipboard
	A	Rate selected markers as "approved"
	R	Rate selected markers as "rejected"
	U	Rate selected markers as "uncertain"
		Trending and Spectrogram
Ctrl	S	Show spectrogram
Ctrl	T	Show trending
		Edit Comment
Ctrl	E	Open the comment dialog
Ctrl	Enter	Close patient comment dialog and save comment modifications
	Esc	Close patient comment dialog and discard comment modifications
		Spectrogram and Trending Widget
Ctrl	Arrow left	Scroll spectrogram/trending plot one page to the left

Ctrl	Arrow right	Scroll spectrogram/trending plot one page to the right
Ctrl + Shift	Arrow left	Scroll spectrogram/trending plot 30 seconds to the left
Ctrl + Shift	Arrow right	Scroll spectrogram/trending plot 30 seconds to the right

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- Contenuti generati automaticamente**: Points to the right-hand side of the report, which contains patient information, statistical data, and a seizure period chart.
- Compila**: Points to the left-hand side of the report, which contains input fields for 'Indication for testing', 'Summary of the findings', 'Diagnostic significance', and 'Clinical comments'.
- Copia commento automaticamente**: Points to the 'Paste from EpiSight comment' button located between the 'Diagnostic significance' and 'Clinical comments' sections.
- Annulla**: Points to the 'Clear All' button in the top right corner.
- Esporta in pdf**: Points to the 'Create PDF' button in the top right corner.

The report content includes:

- Patient information**: Date of birth (1970-01-01), Age at study time (49), Sex (unselected), Treating physician (unselected), Seizure types (Implant ID: E 02, Location: Unknown).
- Statistical information**:

Hours recorded: 967 630	Electrographic seizures: 0 0
Usage total: 84% 56%	Electrographic seizure rate per 24 hours: 0.00 0.00
Usage day (7am - 10pm): 84% 56%	Electrographic seizure rate per 24 hours of recording: 0.00 0.00
Usage night (10pm - 7am): 83% 55%	Patient-reported events: 0 0
- Seizure periods**: A chart showing seizure activity over time, with a legend for Weekend data, Weekdays data, No data, and Electrographic seizure.

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The screenshot shows the UNEEG EpiSight Analyzer interface with the following components and annotations:

- Top Bar:** Contains the UNEEG logo, a search bar, and buttons for 'Clear search', 'Select...', and 'Create new patient'.
- Patients Table:** A table with columns: Patient ID, Last name, First name, Birth date, Connection, and Open. It lists two patients: JM19700 (John Doe, 1970-01-01, Unconnected) and JM19800 (Jane Smith, 1980-01-01, Unconnected).
- Details for Doe John - Unconnected:** A detailed view of a patient's information, including:
 - Patient information:** Patient ID, Last name, First name, Birth date, Sex, Seizure type(s), Treating physician, Implant serial no., and Location.
 - Data information:** Start of recording, Current end of recording, Available amount of data, New data since last review, and Reviewed until.
 - Data overview:** A timeline showing data availability (grey), no data available (white), and data analyzed (blue).
- Annotations (Arrows pointing to interface elements):**
 - Buscar paciente:** Points to the search bar.
 - Borrar búsqueda:** Points to the 'Clear search' button.
 - Seleccionar columnas:** Points to the 'Select...' button.
 - Añadir nuevo paciente:** Points to the 'Create new patient' button.
 - Comprobar conexión:** Points to the 'UNEEG ProConnect' button.
 - Cambiar los ajustes:** Points to the 'Settings' button.
 - Abrir ayuda:** Points to the 'Help' button.
 - Abrir datos:** Points to the 'Open patient' button.
 - Generar informe:** Points to the 'Report' button.
 - Importar datos:** Points to the 'Connect' button.
 - Editar detalles:** Points to the 'Import manually' button.
 - Añadir comentario:** Points to the 'Comment' text area.
 - Leer información:** Points to the 'Data information' section.

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