**Quick Start Guide** **EN**

**Schnelleinstieg** **DE**

**Guida rapida** **IT**

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## Quick Start Guide

### UNEEG EpiSight Analyzer

Version number 2.4

Manufacturer:  
AIT - Austrian Institute of Technology GmbH;  
Giefinggasse 4; 1210 Vienna; Austria  
Date of manufacture: 2025

Use-by date:  
Windows 10: 10/2025  
Windows Server 2019: 10/2025  
Windows 11: 10/2027

Notified Body:  
mdc medical device certification GmbH;  
Kriegerstrasse 6; 70191 Stuttgart; Germany

Medical device

Caution: Please read paragraphs marked with this symbol carefully as they contain important safety information.

(01)09120109830276

Issuing date of this document:  
2025-03

## Table of Content

Table of Content.....	3
Warnings and precautions.....	4
Contraindications.....	5
Possible adverse effects.....	5
Quick Start Guide .....	6
Installation and license activation .....	7
Patient Database .....	8
Trending .....	9
Spectrogram.....	10
Report .....	11
Keyboard shortcuts.....	12

## Warnings and precautions

UNEEG EpiSight Analyzer is CE-certified as a medical device. If your country requires any other approval for medical devices you must not use UNEEG EpiSight Analyzer.

The modules from UNEEG EpiSight Analyzer for the automatic analysis of EEG cannot replace the examination by the physician. As for any other automatic procedure, there can be inaccuracies during the analysis with UNEEG EpiSight Analyzer. The original EEG still needs to be used for the evaluation and the results of UNEEG EpiSight Analyzer need to be confirmed based on the unaltered raw EEG trace.

UNEEG EpiSight Analyzer does not provide any diagnosis or diagnostic recommendation, conclusion or prediction of the patient's state. It remains the responsibility of the physician to decide on the diagnosis or to induce treatment.

The use of the software cannot replace the real-time surveillance by medical staff particularly of the vital functions. The patient's safety remains the responsibility of the medical professional.

The maximum length for continuous EEG recording that can be reviewed and analyzed with the EEG viewer is 5 years. Any recording that starts later than 5 years after the first recording will not be processed by UNEEG EpiSight Analyzer.

For UNEEG EpiSight Analyzer seizure detection, detection of rhythmic pattern, aEEG, background frequency and sleep analysis the EEG of at least three subcutaneous electrodes must be available.

UNEEG EpiSight Analyzer seizure detection cannot be used as an alarm system, since there may be a delay of several minutes between the clinical onset of a seizure and the display of the detected pattern. Some EEG-events with epileptic seizures that have an electrographic correlate might be missed. For this reason UNEEG EpiSight Analyzer can only support but can not replace the qualified expert in the real-time monitoring.

UNEEG EpiSight Analyzer detection of rhythmic pattern, aEEG, background frequency or sleep analysis included in UNEEG EpiSight Analyzer should not be used as an alarm system, since the visualization of patterns could be delayed by up to several minutes.

Some events, where patterns defined in the Standardized Critical Care EEG Terminology of the American Clinical Neurophysiology Society are visible in the EEG, might be missed. All measures of UNEEG EpiSight Analyzer must be evaluated in combination with the original raw EEG and must not be the only basis for a diagnosis, treatment or change of treatment.

## Contraindications

There are no known contraindications for the use of UNEEG EpiSight Analyzer.

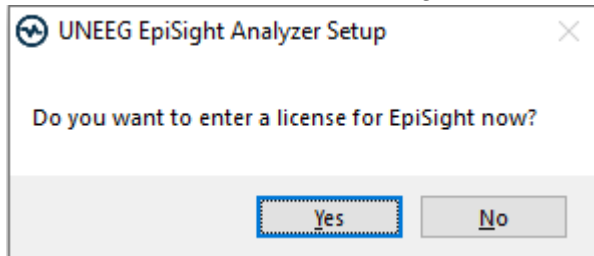
## Possible adverse effects

There are no known possible adverse effects when using UNEEG EpiSight Analyzer.

# Quick Start Guide


## Installation and license activation

1. Download the installer for UNEEG EpiSight Analyzer.
2. Start the installer by double-clicking the downloaded executable.
3. The installer will guide you through the installation process.
4. After the installation the following window will pop up:


If you click “Yes” you will proceed at step 9.

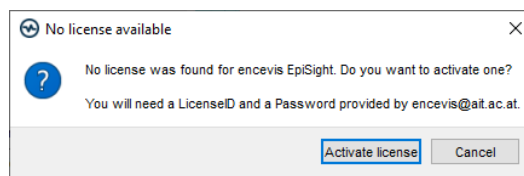5. Once the installation has finished, start the EpiSight Analyzer by clicking on the icon on your desktop.
6. You can add new patients to the database and import data.
7. The following steps are just necessary if you have no valid License activated so far. In order to open the patient’s EEG, you will need an active license.
8. Click on “*Open patient*”, the following window will pop up:

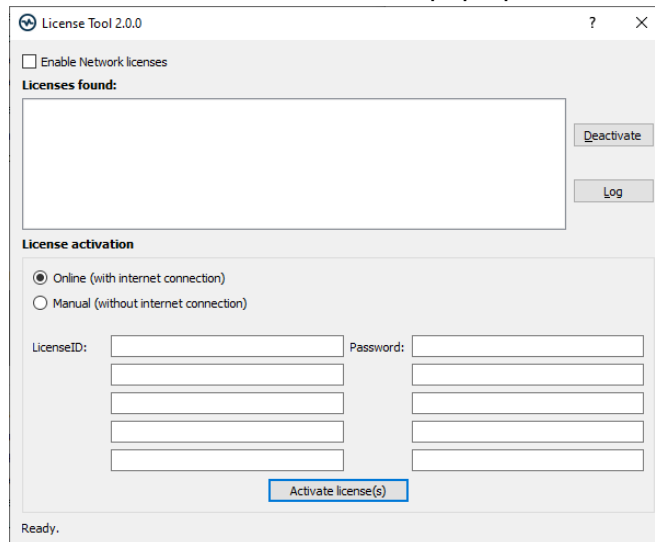9. Press “*Activate license*”, the License Tool will pop up.

10. If you are connected to the internet, select „*Online*“ as activation mode. Otherwise, select “*Manual*” and contact your distributor in order to get the password for the manual activation.
11. Enter your LicenseID and Password.
12. Press “*Activate license(s)*”.

## Patient Database

Starting UNEEG EpiSight Analyzer will open the patient database window.

The screenshot shows the UNEEG EpiSight Analyzer interface. The top navigation bar includes 'Search Patient', 'Clear search', 'Select columns', 'Add new patient', 'Check connection', 'Change settings', and 'Open Help'. The main area is divided into a 'Patients' list on the left and a 'Details for Michael Jones - Connected' panel on the right. The 'Patients' list has columns for Patient ID, Last name, First name, Birth date, Connection, and Open. The 'Details' panel includes 'Patient information', 'List of UNEEG devices', 'Comment', 'Data information', and 'Data overview'. The 'Data overview' shows a timeline with 'data available', 'no data available', and 'data analyzed' indicators. Arrows point from labels to these specific features: 'Search Patient', 'Clear search', 'Select columns', 'Add new patient', 'Check connection', 'Change settings', 'Open Help', 'Open data', 'Generate report', 'Import data', 'Edit details', 'Add comment', and 'Read data information'.

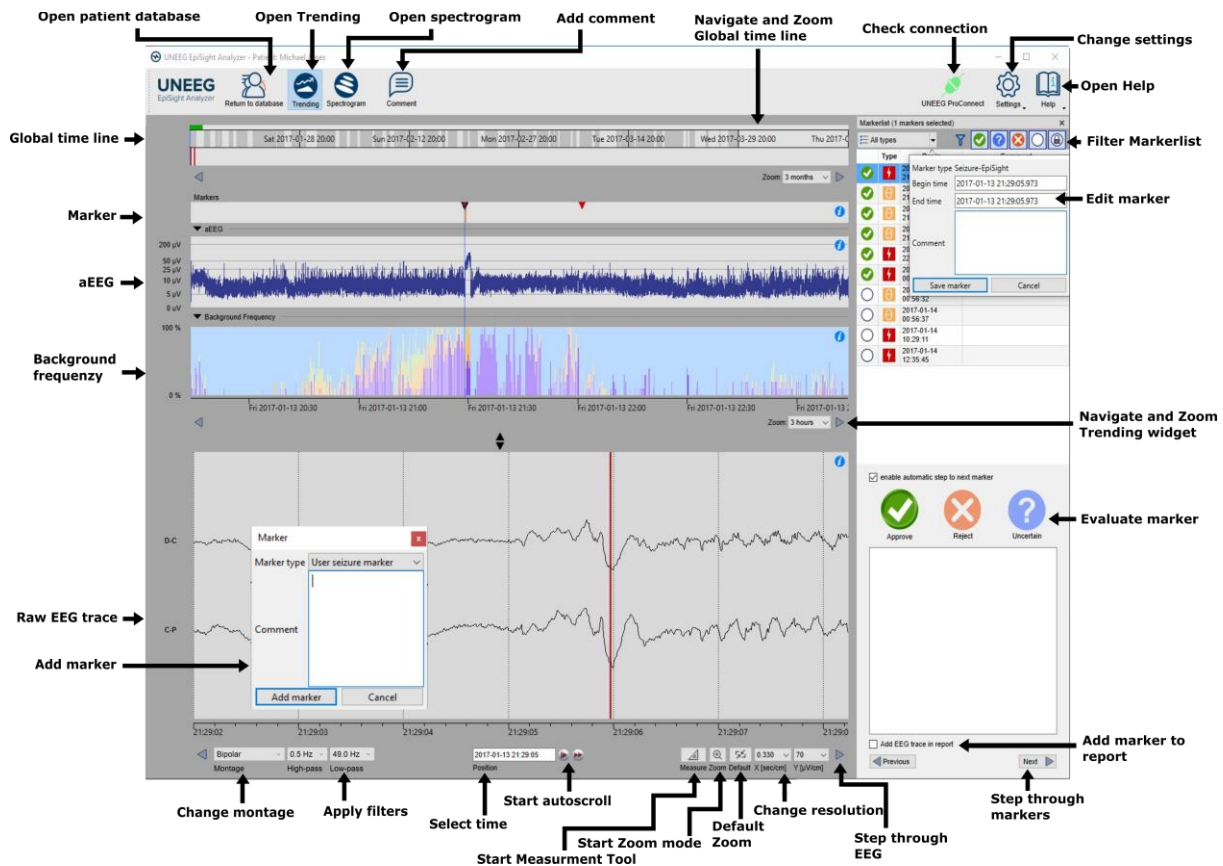
The patient database offers the following functionalities:

- Search for a patient by ID, name or date of birth
- Select the columns shown in the patient list
- Add a new patient
- Edit the patient's information
- Get the information about the EEG data available for each patient
- Open the patient's EEG
- Generate a report
- Connect to UNEEG ProConnect
- Import data by connecting to UNEEG ProConnect
- Import data manually by selecting a folder
- Change settings
- Open help

## Trending

Clicking on “Open patient” will open the trending window.


The trending includes the following functions and traces:

- Global timeline with an overview over all available data
- Zoom in the global timeline and in the analysis windows over a drop-down menu or the mouse wheel
- Seizure and other markers in a trace and in a list on the right side of the window
- Rhythmic pattern markers
- Amplitude-integrated EEG (aEEG)
- Background frequency
- EEG plot synchronized to the traces above
- Change montage
- Apply notch, high-pass and low-pass filters
- Change resolution in time and voltage
- Apply standard zoom
- Start zoom mode
- Start measurement Tool
- Start autoscroll
- Add markers
- Review and edit markers
- Filter marker list by marker type and by status

## Spectrogram

Clicking on the icon “*Spectrogram*” will show the spectrogram traces.

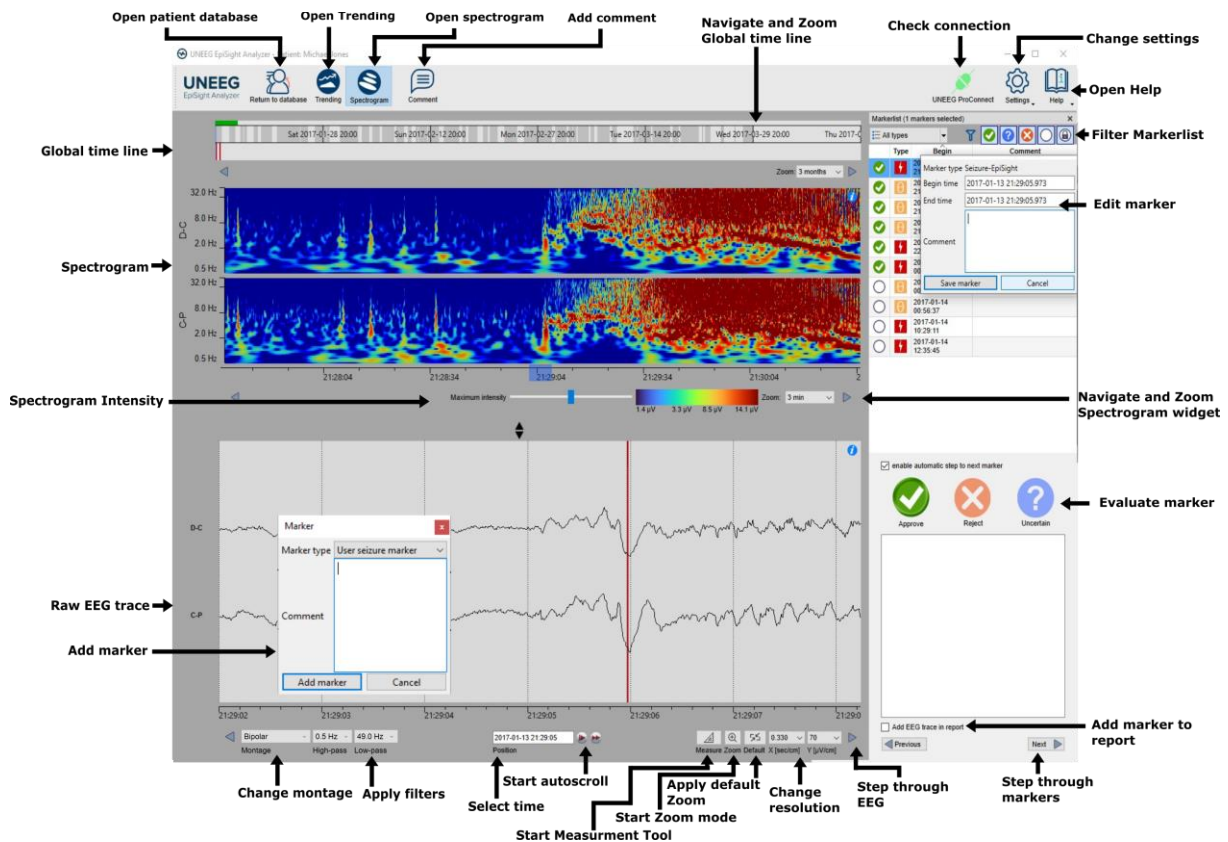The spectrogram adds the following functionalities:

- Display of the spectrogram of the channels D-C and C-P.
- Get the estimated frequency in the spectrogram trace
- Adjust the color settings by changing the maximum intensity shown.

## Report

Clicking on “Report” will open the report window.

The screenshot shows the UNEEG EpiSight Analyzer Report interface. Key elements include:

- Report Section:** Fields for 'Reported by', 'Reference period' (start/end: 2017-01-13 to 2017-02-20), and 'Current period' (start/end: 2017-02-21 to 2017-03-29).
- Patient Information:** Name: Michael Jones, Sex: unselected, Date of birth: 1970-01-01, Age at study time: 47, Treating physician: unselected, Seizure Types: unselected, Implant ID: E 06, Location: Left.
- Recording and report summary:**

	Reference period	Current period
Days of recording	39	37
Usage (%)	62%	61%
Electrographic seizures	4	0
Electrographic seizure rate per 24 hours	0.10	0.00
Patient-reported events	0	0
- Seizures over time:** A bar chart showing seizure activity over 24 hours, with a legend for 'Weekend data', 'Weekdays data', 'No data', 'Electrographic seizure', and 'Patient-reported events'.

The report offers the following functionalities:

- Enter additional clinical information including:
  - Indication
  - Summary of the findings
  - Diagnostic significance
  - Clinical comments
- Generate content automatically including:
  - Patient information
  - Statistical information
  - Overview of seizure periods over time
  - Seizure periodicity plots
  - Seizure rate over time
  - Examples of seizures
- Paste the EpiSight comment
- Export to a pdf file

## Keyboard shortcuts

The following table shows a summary of keyboard shortcuts available

Modifier	Key	Function
		<b>EEG Plot area</b>
	Arrow left	Scroll EEG-Plot one page to the left
	Arrow right	Scroll EEG-Plot one page to the right
Shift	Arrow left	Scroll EEG-Plot one second to the left
Shift	Arrow right	Scroll EEG-Plot one second to the right
	Home	Scroll EEG-Plot to the beginning of the recording
	End	Scroll EEG-Plot to the end of the recording
Ctrl	+	Zoom one step into the EEG Area
Ctrl	-	Zoom one step out of the EEG Area
Ctrl	0	Zoom EEG Area to default value
Shift	MouseWheel	Zoom at the cursor position in EEG Area
Ctrl	M	Enter measurement mode
	Esc	Exit measurement mode
		<b>Marker List</b>
	Arrow up	Scroll one row up in the marker list
	Arrow down	Scroll one row down in the marker list
Shift	Arrow up	Expand selection with next marker up
Shift	Arrow down	Expand selection with next marker down
Ctrl	A	Select all marker currently visible in the marker list
Ctrl	C	Copy selected marker to the clipboard
	A	Rate selected markers as "approved"
	R	Rate selected markers as "rejected"
	U	Rate selected markers as "uncertain"
		<b>Trending and Spectrogram</b>
Ctrl	S	Show spectrogram
Ctrl	T	Show trending
		<b>Edit Comment</b>
Ctrl	E	Open the comment dialog
Ctrl	Enter	Close patient comment dialog and save comment modifications
	Esc	Close patient comment dialog and discard comment modifications

		<b>Spectrogram and Trending Widget</b>
Ctrl	Arrow left	Scroll spectrogram/trending plot one page to the left
Ctrl	Arrow right	Scroll spectrogram/trending plot one page to the right
Ctrl + Shift	Arrow left	Scroll spectrogram/trending plot 30 seconds to the left
Ctrl + Shift	Arrow right	Scroll spectrogram/trending plot 30 seconds to the right

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The screenshot shows the UNEEG EpiSight Analyzer interface. The top navigation bar includes 'UNEEG ProConnect', 'Settings', and 'Help'. The main area is divided into a 'Patients' table on the left and a 'Details for Michael Jones - Connected' panel on the right. The 'Patients' table has columns for Patient ID, Last name, First name, Birth date, Connection, and Open. The 'Details' panel shows patient information, a list of UNEEG devices, a comment field, and data information including recording start/end times and a data overview chart.

Callouts point to the following features:

- Ricerca paziente**: Search bar in the Patients table.
- Annulla ricerca**: Clear search button.
- Seleziona colonne**: Select dropdown menu.
- Aggiungi nuovo paziente**: Create new patient button.
- Verifica connessione**: UNEEG ProConnect status indicator.
- Modifica delle impostazioni**: Settings icon.
- Apri supporto**: Help icon.
- Apri dati**: Open EEG button in the details panel.
- Genera un rapporto**: Open Report button.
- Importa dati**: Import manually button.
- Modifica dettagli**: Edit icon in the patient information section.
- Aggiungi commento**: Comment text area.
- Leggi informazioni sui dati**: Data information section.

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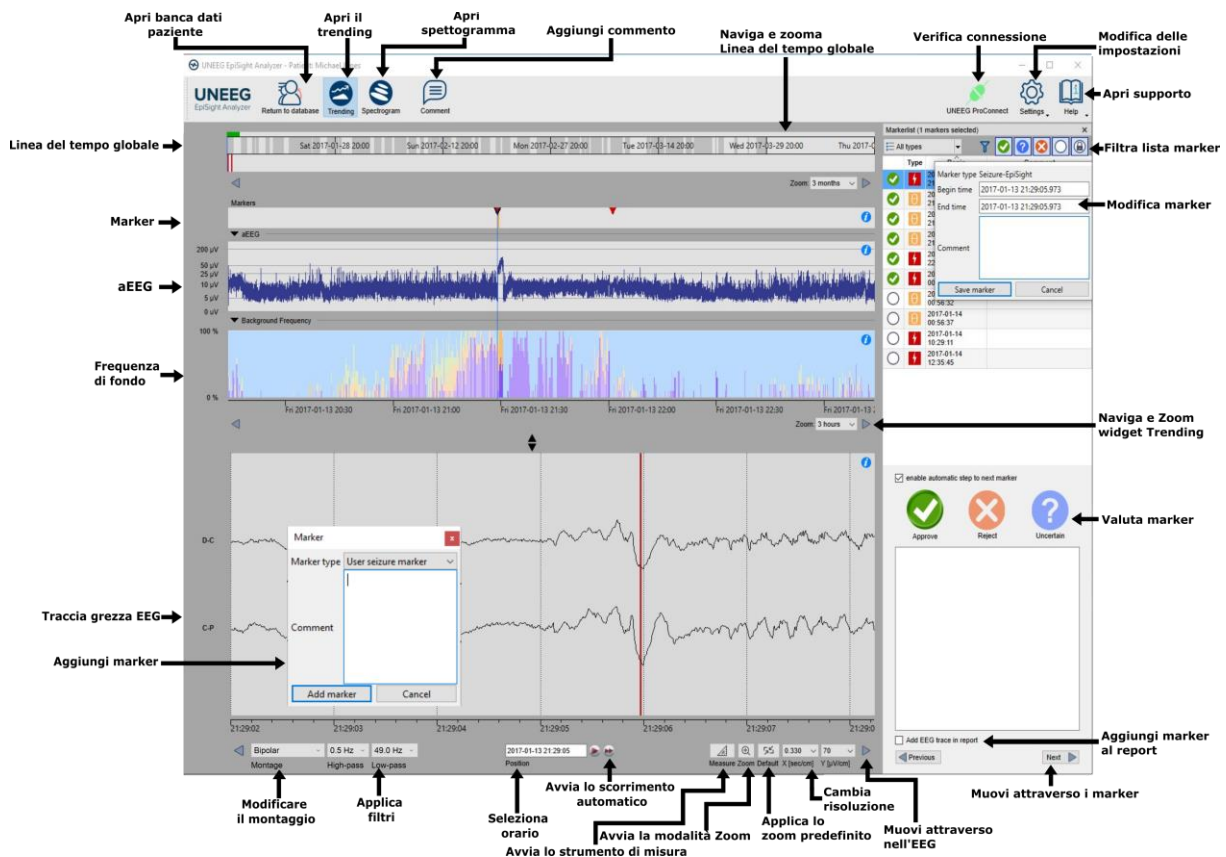
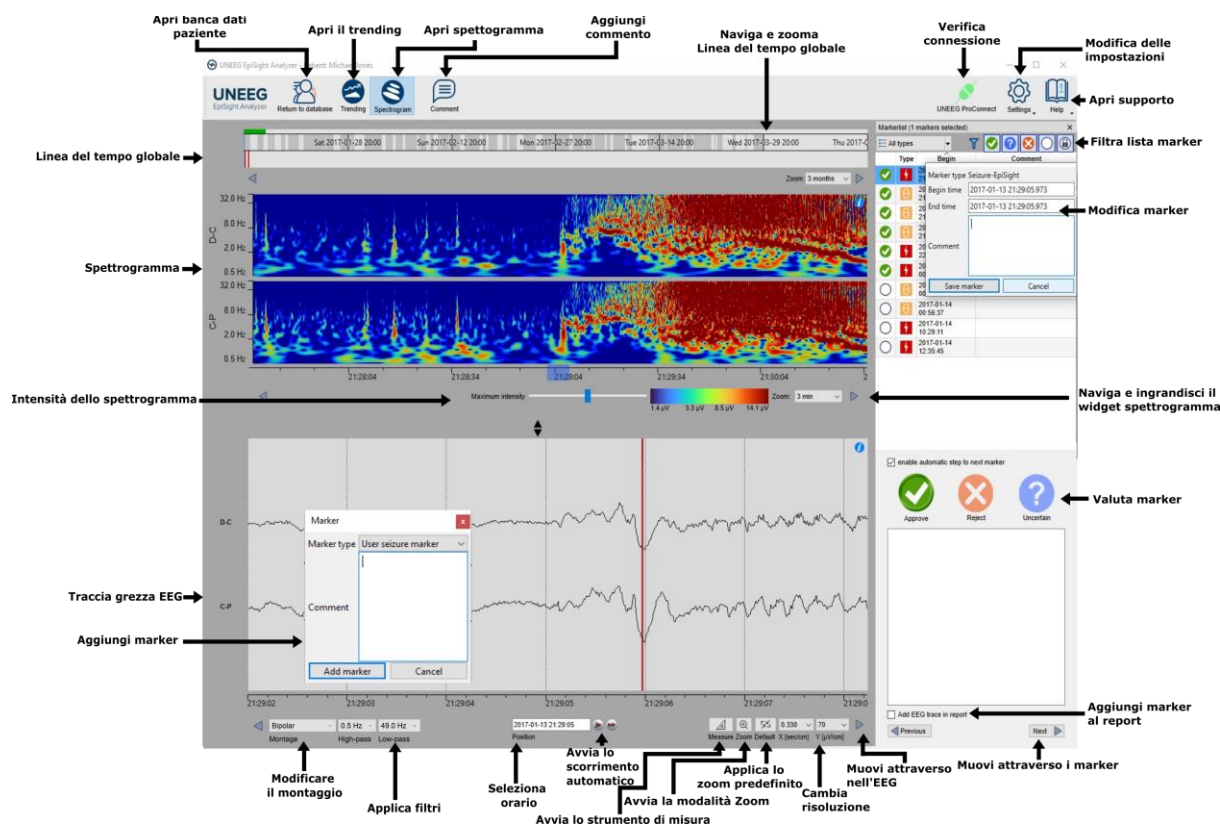
The screenshot shows the UNEEG EpiSight Analyzer report interface. Annotations include:

- Contenuti generati automaticamente**: Points to the Patient Information and Recording and report summary sections.
- Annulla**: Points to the 'Clear All' button in the top right.
- Esporta in pdf**: Points to the 'Create PDF' button in the top right.
- Compila**: Points to the 'Reported by', 'Summary of the findings', 'Diagnostic significance', and 'Clinical comments' fields.
- Copia commento automaticamente**: Points to the 'Paste from comment' button.

The report content includes:

- Reported by**: Form field.
- Reference period**: 2017-01-13 to 2017-02-20.
- Current period**: 2017-02-21 to 2017-03-29.
- Patient Information**: Name: Michael Jones, Date of birth: 1970-01-01, Age at study time: 47, Location: Left.
- Recording and report summary**:
 

	Reference period	Current period
Days of recording	39	37
Usage (%)	62%	61%
Electrographic seizures	4	0
Electrographic seizure rate per 24 hours	0.10	0.00
Patient-reported events	0	0
- Seizures over time**: A heatmap showing seizure activity over a 24-hour period, with a legend for Weekend data, Weekdays data, No data, Electrographic seizure, and Patient-reported events.

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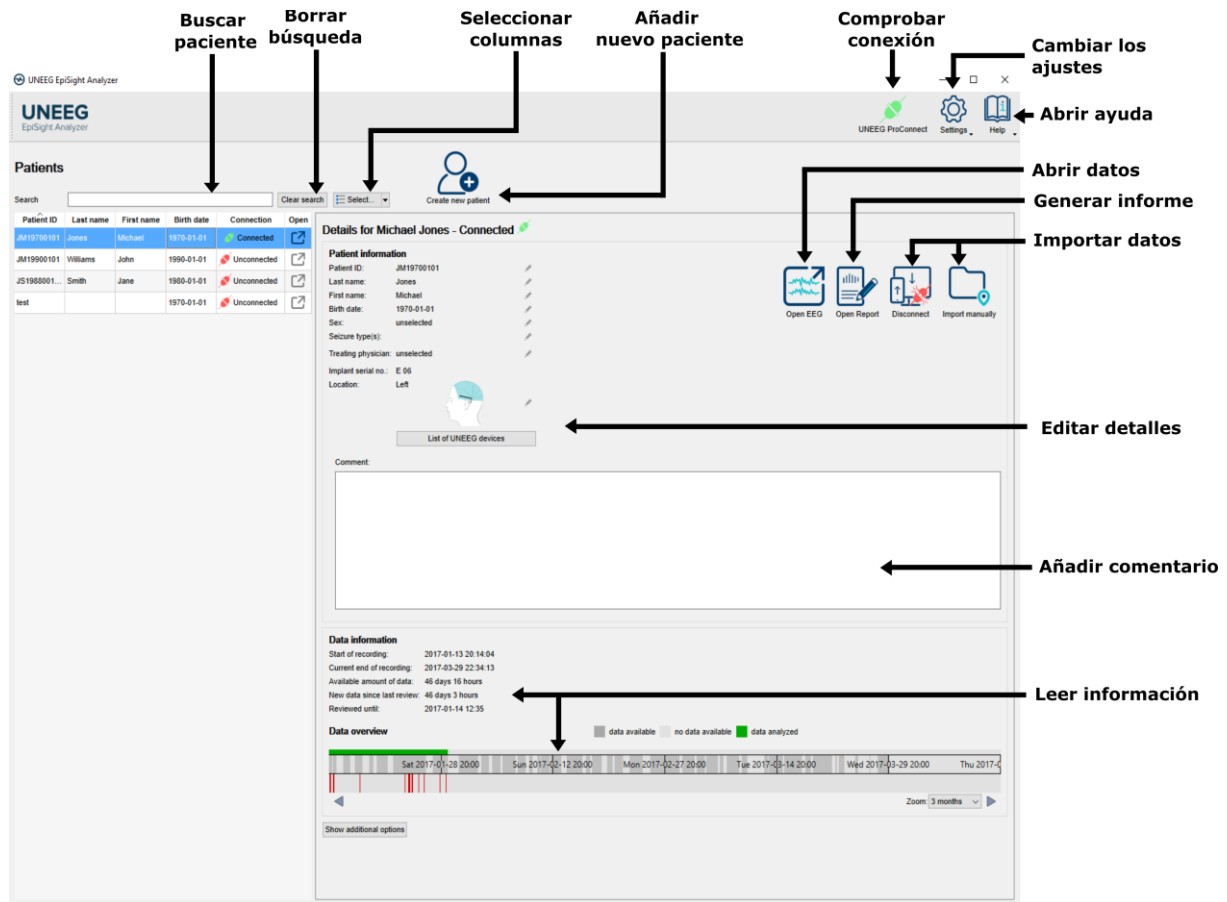
The screenshot shows the UNEEG EpiSight Analyzer interface. At the top, there are navigation icons for 'UNEEG ProConnect', 'Settings', and 'Help'. Below this is a 'Patients' section with a search bar and a 'Create new patient' button. A table lists patients with columns for Patient ID, Last name, First name, Birth date, Connection status, and an 'Open' button. The 'Details for Michael Jones - Connected' view is shown, containing 'Patient information' (Last name, First name, Birth date, Sex, Seizure type(s), Treating physician, Implant serial no., Location), a 'List of UNEEG devices', a 'Comment' field, and 'Data information' (Start of recording, Current end of recording, Available amount of data, New data since last review, Reviewed until). A 'Data overview' chart shows data availability and analysis over time. Labels with arrows point to various features: 'Buscar paciente' (Search patient), 'Borrar búsqueda' (Clear search), 'Seleccionar columnas' (Select columns), 'Añadir nuevo paciente' (Add new patient), 'Comprobar conexión' (Check connection), 'Cambiar los ajustes' (Change settings), 'Abrir ayuda' (Open help), 'Abrir datos' (Open data), 'Generar informe' (Generate report), 'Importar datos' (Import data), 'Editar detalles' (Edit details), 'Añadir comentario' (Add comment), and 'Leer información' (Read information).

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